



# TR Comfortline Bathtub

*Hygiene Equipment  
Designed for People*

Spare part list

CE




Registered at Medical Products  
Agency, Sweden,  
as a MD Class 1 product,  
MDD 93/42EEC.

TR reserves the right to make  
design changes and/or  
improvements without notice.






**TR EQUIPMENT**

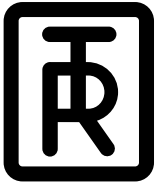

# TR Comfortline Bathtub

Hygiene Equipment  
Designed for People

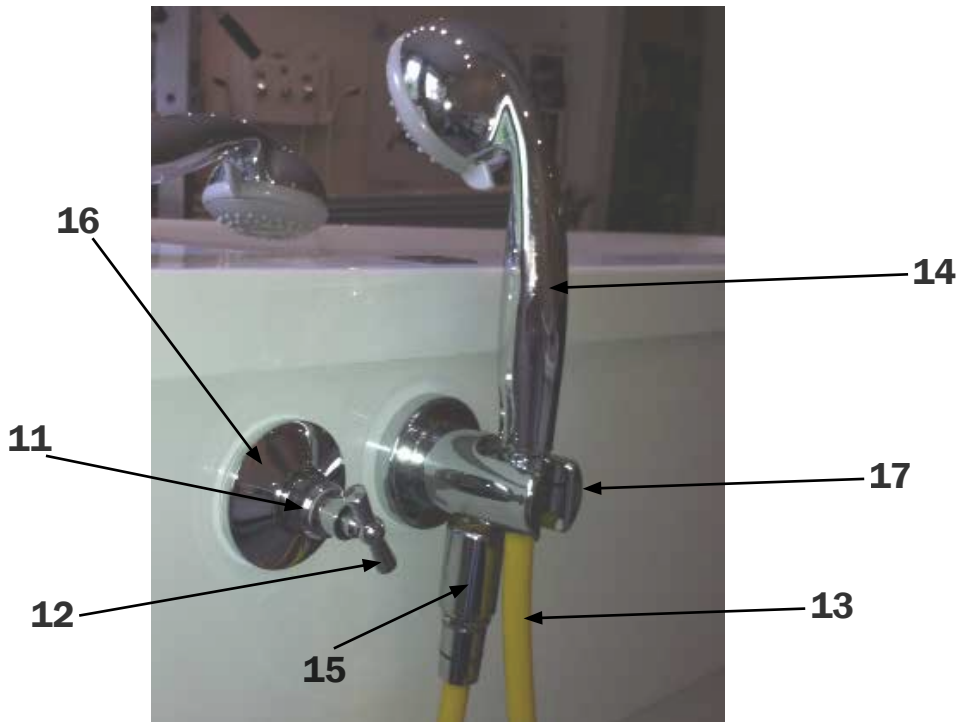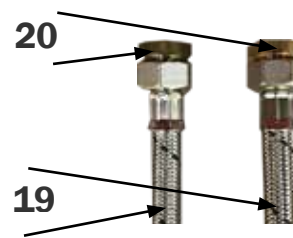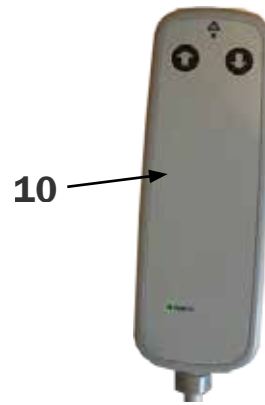


**Head office**  
TR Equipment AB, Sweden  
Phone: +46 140 38 50 60  
info@trequipment.se  
[www.trequipment.com](http://www.trequipment.com)

**Group Subsidiaries**  
TR Group Inc, US, Phone: +1 (800) 752 6900  
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386  
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



**TR EQUIPMENT**

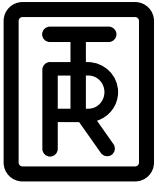

# TR Comfortline Bathtub

Hygiene Equipment  
Designed for People



Pos.	Art.no.	Description
1	829280	Overflow, plug and drain
2	851840	Foot bathtub
3	851861	Footsupport
4	851890	Actuator
	851892	Controlbox for actuator, 230V
	851893	Controlbox for actuator, 230V (replaces 851892)
5	829173	Bracket for shower handle
6, 14	851868	Shower handle, chrome
7	851867	Shower hose chrome
8	851871	Thermostat
9	851872	Vertical connectors for thermostat
10	851894	Handcontrol
10	851894-03	Handcontrol (replaces 851894)
18	829279	Loose plug for drainage
19	851880	Hose hot/cold water L=1,5m
20	820198 6090	Conversion nippel R20/R15

**Handcontrols 851894 and 851894-03  
fits controlboxes 851892 and 851893.**

### Special parts for Cleaning system

Pos.	Art.no.	Description
	829070	Injector cleaning Dema
11	829289	Cleaning valve incl. Key
12	829289-02	Key for cleaning valve
13	829096	Shower hose, yellow
6, 14	851868	Shower handle, chrome
	851873	Lid, cleaning bottle
15	851875	Pipe Interrupter
16	851876	Escutcheon, chrome
17	851877	Bracket for shower handle, cleaning

### Spare parts for following options:

- Bluetooth/AUX music system
- MP3 music
- Air Spa
- LED light system
- Autofill

Send your request for spare parts together with  
the TR serial number

#### Head office

TR Equipment AB, Sweden  
Phone: +46 140 38 50 60  
info@trequipment.se  
[www.trequipment.com](http://www.trequipment.com)

#### Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900  
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386  
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



**TR EQUIPMENT**