Ropox Puslebord Maxi 2

Brugermanual

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Usermanual

Mounting Instruction

This manual need to follow the product

Contents

Product information.....	26
Item number	26
Technical data	27
Daily use.....	27
Recommendations for use.....	28
Important information before mounting.....	29
Placement of 230V socket, hot and cold water and drain.....	29
Receiving the nursing table	30
Dimension sketch	30
1. Mounting to the wall.....	31
2. Mounting of the lower cover	33
3. Mounting of the upper cover.....	33
4. Connection of the worktop.....	34
5. Connection of the safety stop.....	35
6. Connection of water and drain	36
Wiring circuit.....	38
Connections	39
Dismounting.....	39
Labelling.....	40
Label.....	40
Symbols:.....	40
Support.....	41
Troubleshooting.....	41
Waste	41
Factors affecting the product during storage and transport	41
Performance test	42
Spare parts	43
Maintenance	44
Cleaning	44
Claims.....	45

General informations

Product information

It is important to read this manual before mounting and use of the nursing table.

Product : Ropox Nursing table Maxi 2, Heightadjustable

Producer : Ropox, DK-4700 Naestved, Tel. +45 55 75 05 00

The height adjustable nursing table is designed for nursing children up to 75 kg. The nursing table can be used in private homes, daycare centers or similar institutions to ensure correct and optimal working conditions for staff.


The height adjustment is performed by pressing the up and down buttons on the handset.

Item number

Length of the nursing table	Part no.
90cm	50-50672
130cm	50-50670

Nursing table Maxi 2 : 90 cm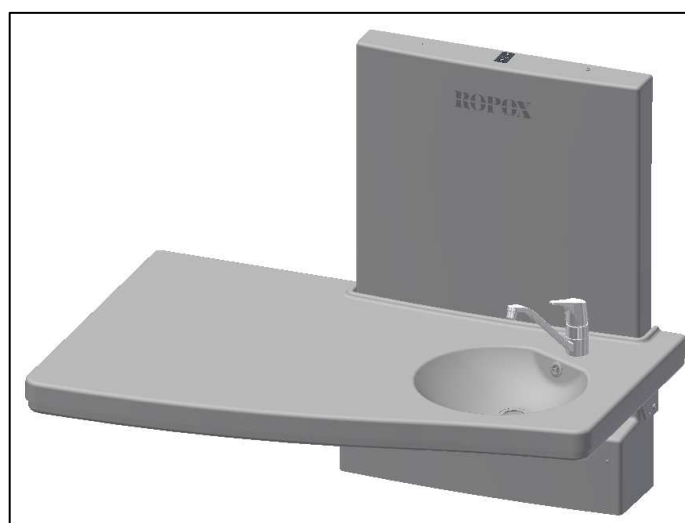

Nursing table Maxi 2 : 130 cm

General informations

Technical data

Power consumption:	230V, 50Hz, max 2.0A
Standby consumption:	7W
Duty cycle:	Maks. 10%, 2min/18min
Height:	Workheight 40 – 100cm Unit B=70cm, H=102cm, D=10/19,8cm
Material – the frame:	Welded steeltubes St.37, stainless steel and aluminum
The worktop:	ABS/PMMA
Surface:	Powder-coated, white RAL 9010 mat, chromate
Max. load:	75 kg


This product has CE marked according to the Machinery, Low Voltage and EMC Directives. It has been tested according to European standards DS / EN ISO 12182 and DS / EN ISO the 60335th

Daily use

Before using this product, it is important that the main power is applied as described under electrical connections.

The height of the nursing table is adjusted by pressing the up and down buttons on the handset.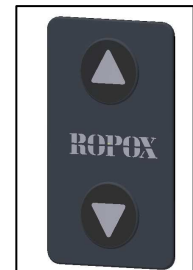

Max. load on the nursing table: 75kg.

Warning !!

During height adjustment of the nursing table, there is a risk of getting jammed between table and adjacent objects.

Before using the table, ensure that there are no objects preventing movement of the table.

!! The product must NOT be operated by children!!

General informations

Recommendations for use

Height adjustment allows individual working height for each employee and can also help to reduce the number of lift when transferring from example: a wheelchair user.

To ensure optimum utilisation of the functions of the nursing table and to avoid accidents, it is important to observe these basic safety procedures.

NEVER leave a child on the nursing table unattended - not even for a short period.

The nursing table should only be used by adults.

Pay special attention to other people who may stay in the room while using the nursing table. Make sure that no children are playing under or with the table.

Make sure that children cannot operate the nursing table unintentionally.

See that there is free space above and below the table to allow height adjustment. Make sure that there are no obstacles above, below or around the nursing table in order to avoid the risk of getting jammed during height adjustment.

If the handset is damaged or does not work, replace it immediately. The buttons are of the "press and keep pressed" type. The motor stops when the press on the button removed. This is an important safety factor.


If two nursing tables are mounted in the immediate vicinity of each other, it is recommended to mount a side cover between them to prevent the risk of getting jammed.

Be constantly aware of objects under the nursing table.

Mounting instruction

Important information before mounting

Make sure that there is free space on all sides of the nursing table as shown below.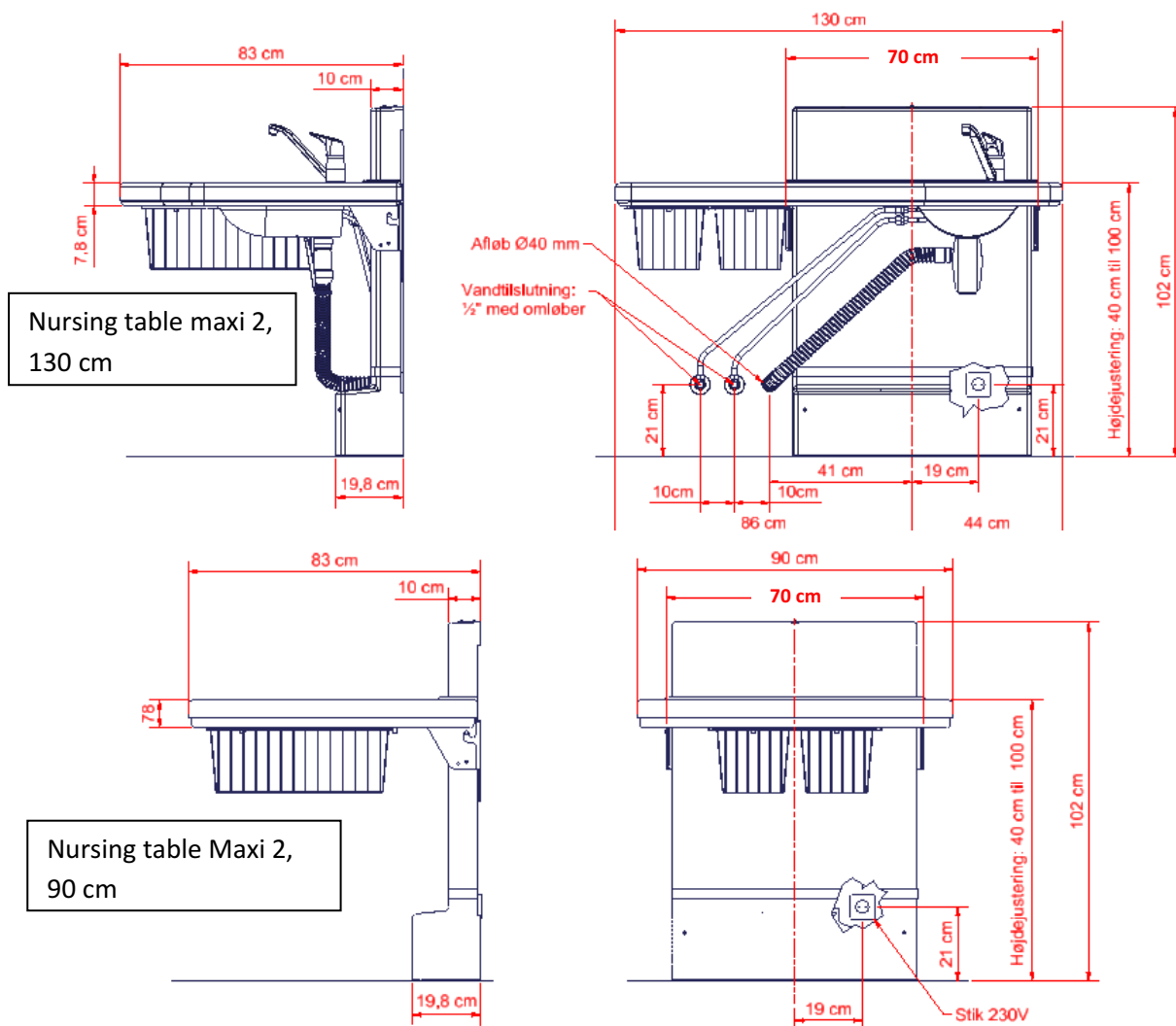

Placement of 230V socket, hot and cold water and drain

If possible, socket for 230V and drain and connection for hot and cold water should be located as shown on the above sketch.

The length of the mains cable is 3,2 m.


Mounting instruction

Receiving the nursing table

These mounting instructions comprise:





Lifting unit	x1	Mounting screws	x7
Covers for the lifting unit	x1	Rawlplugs	x7
Worktop	x1	Bolts M5 x 25	x6
Bolt M4 x 10	x9		
Safety plate	x1	Bolts M8 x 20	x2
Drain and water hose kit	x1	Cable, Safety stop	x1

Dimension sketch
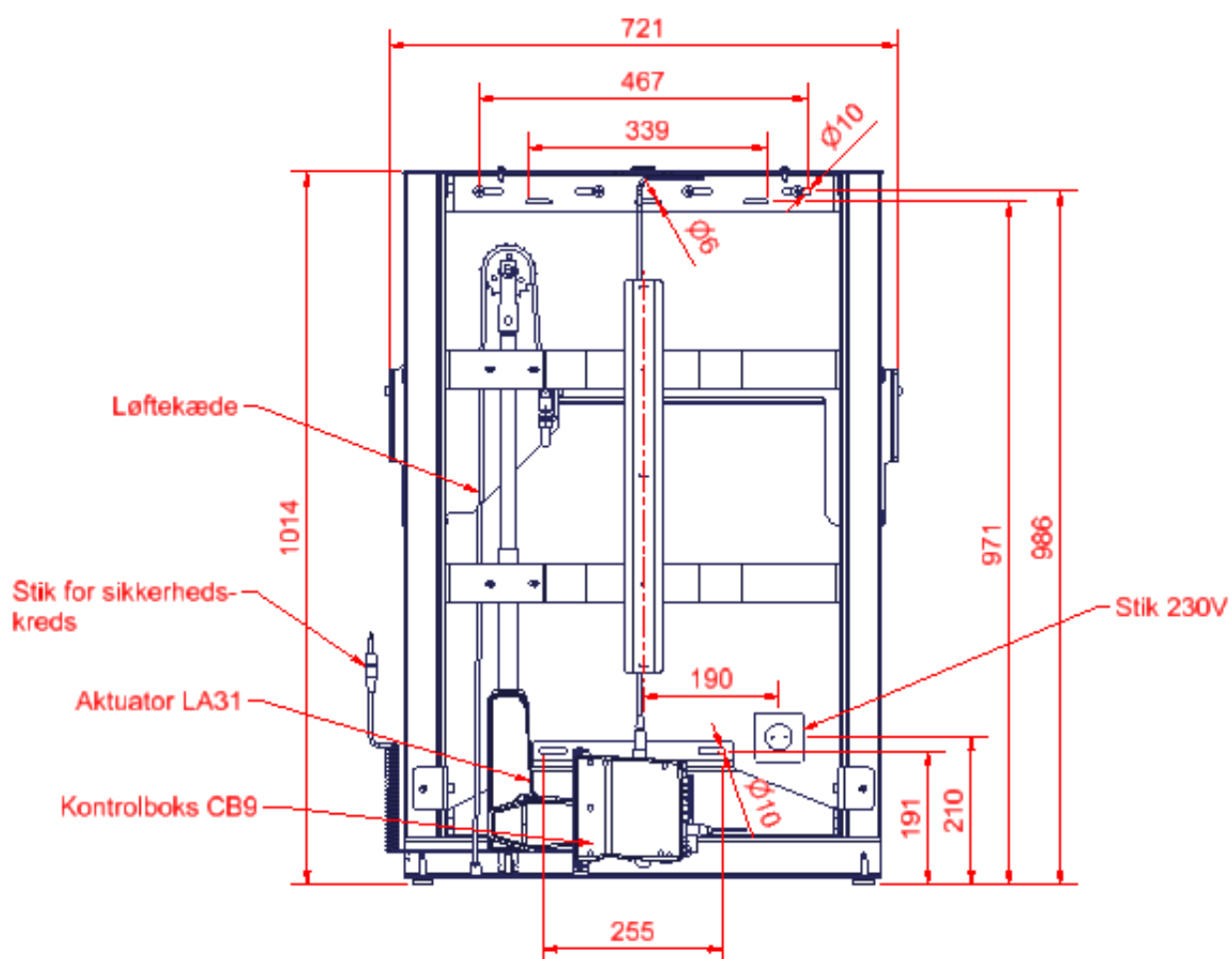


Mounting instruction

1. Mounting to the wall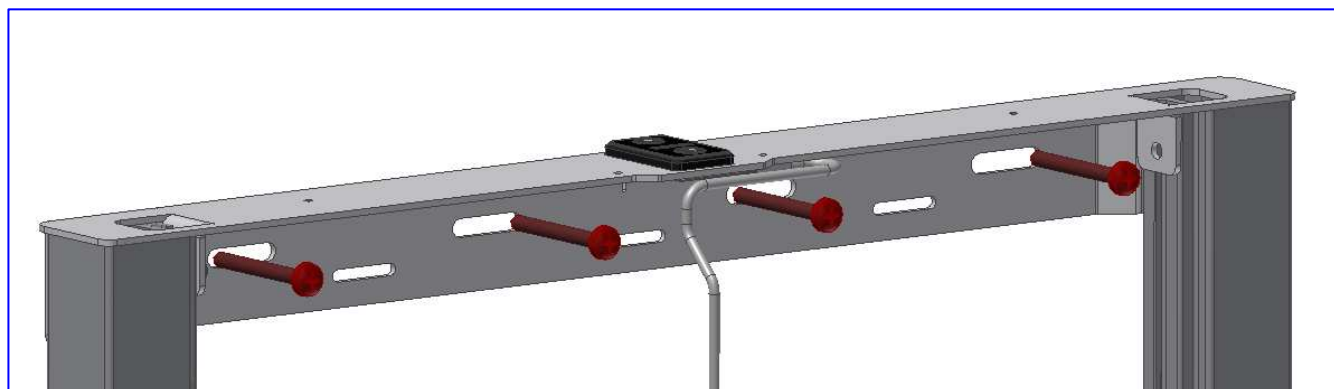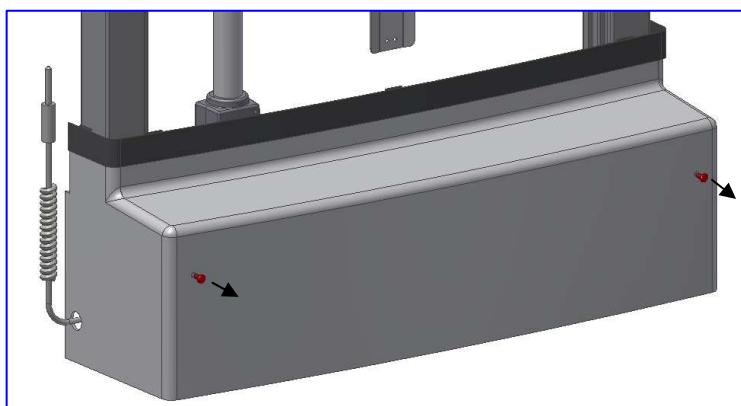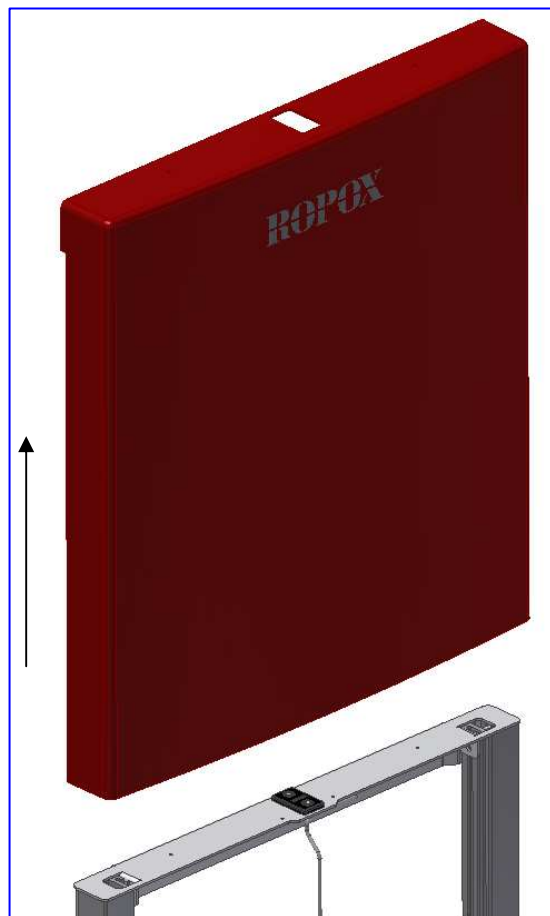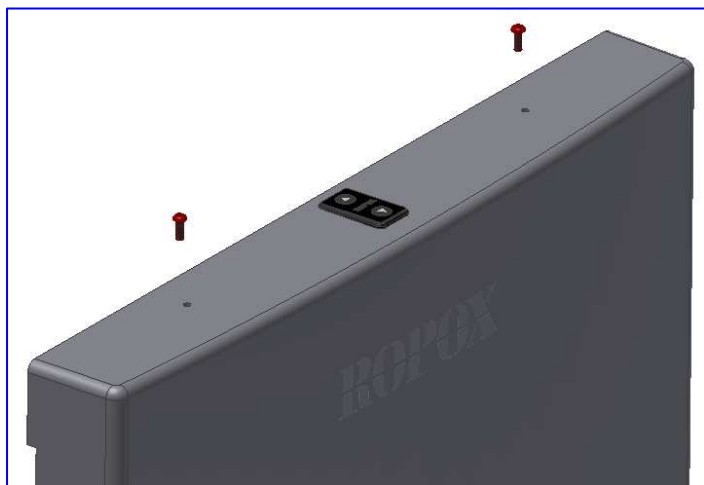


Mounting instruction


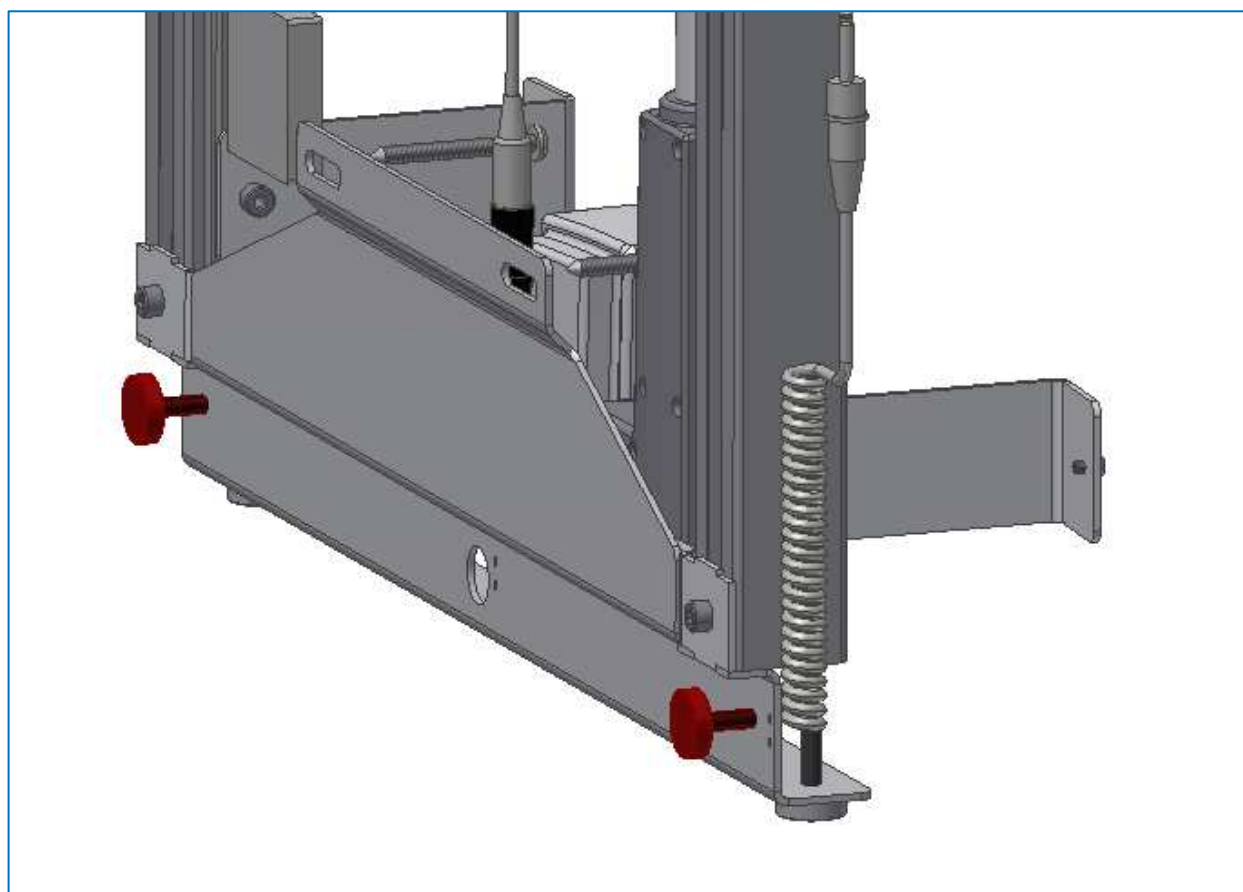
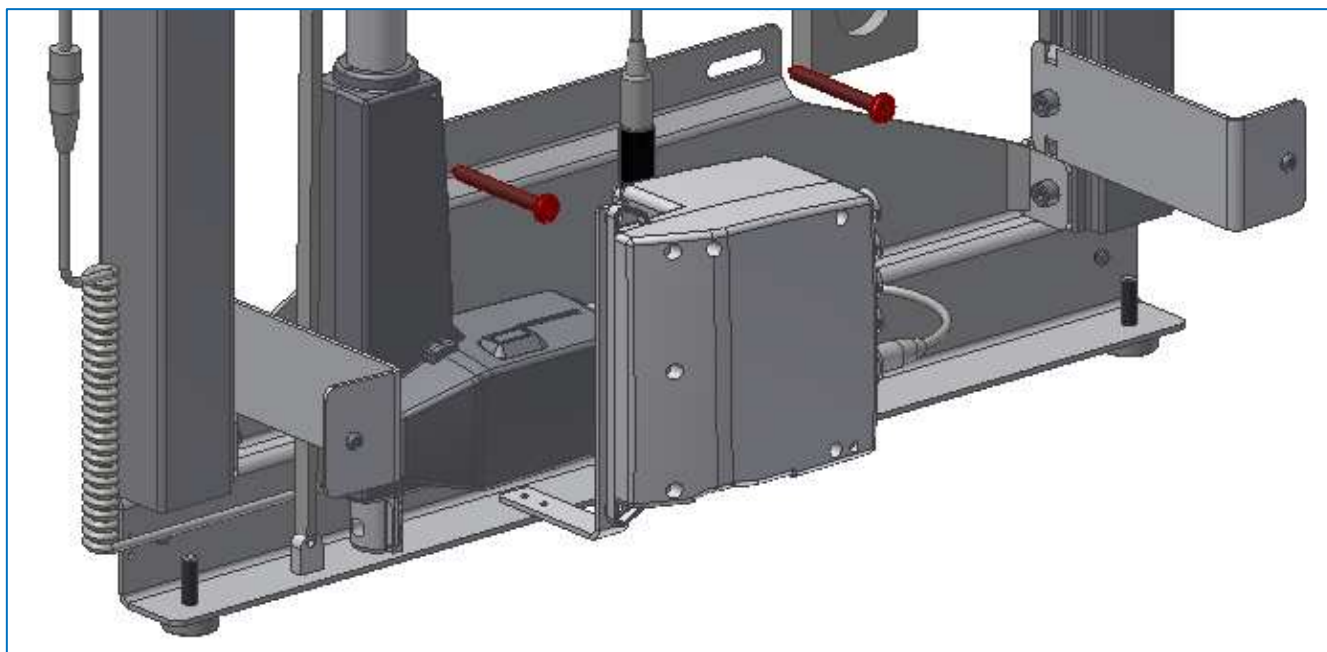
Mounting instruction

2. Mounting of the lower cover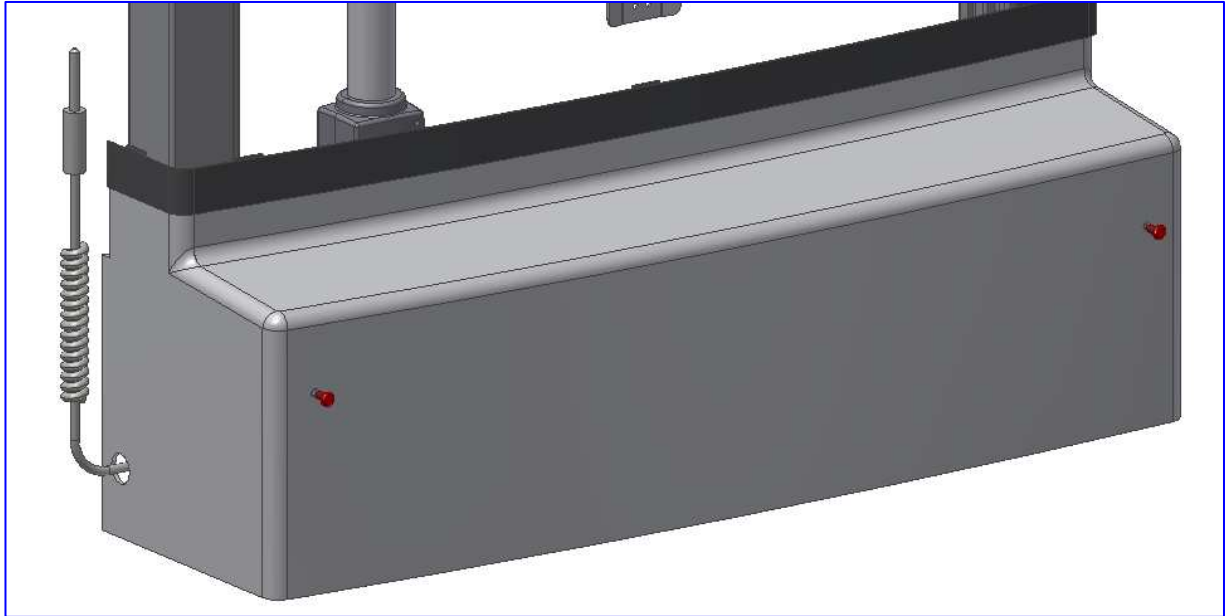




3. Mounting of the upper cover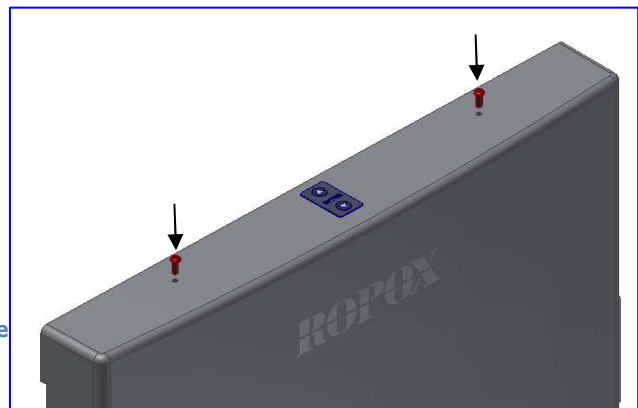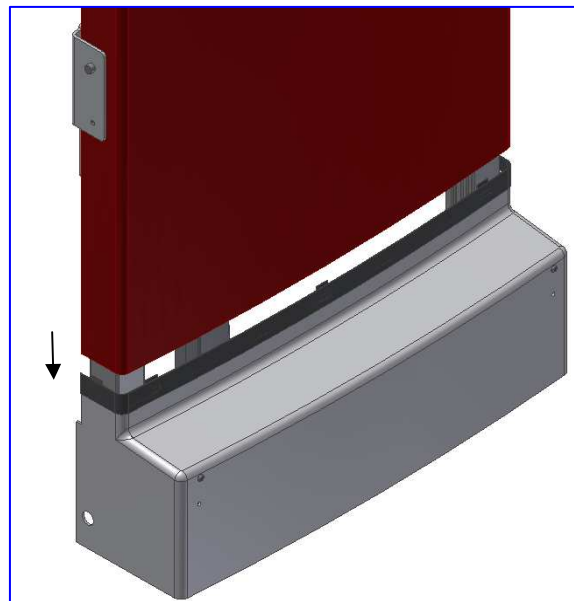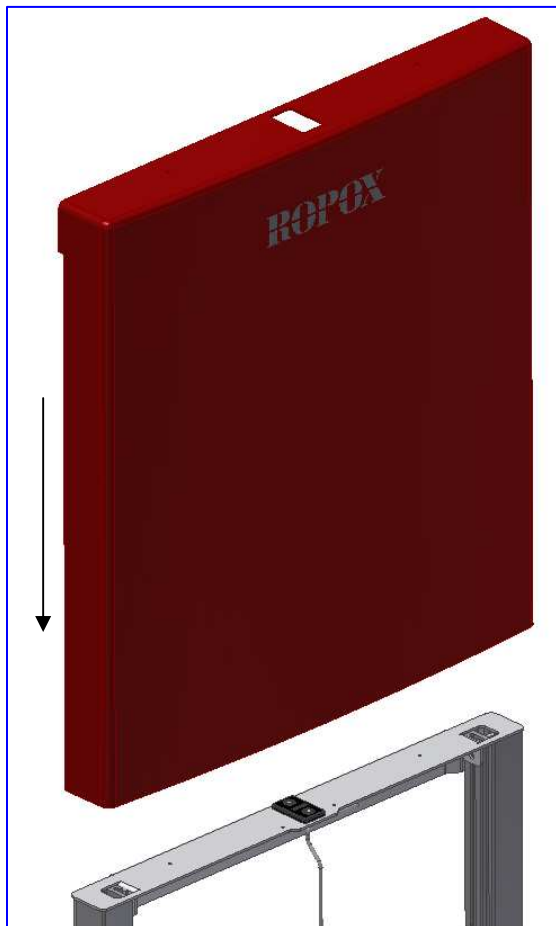

Side


Mounting instruction

4. Connection of the worktop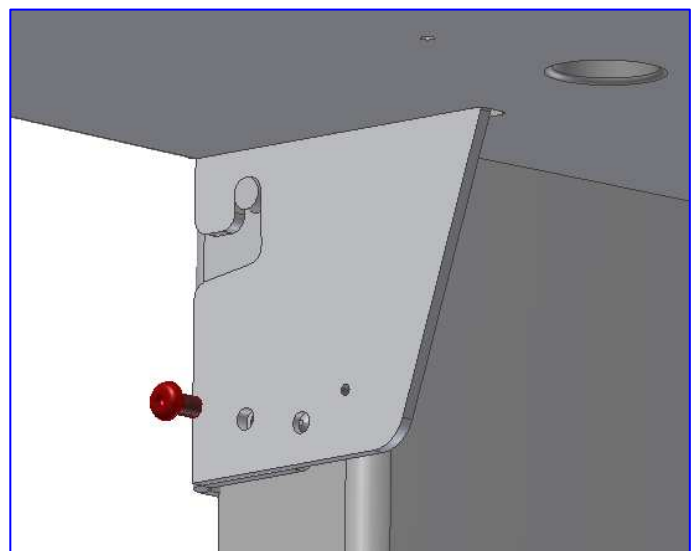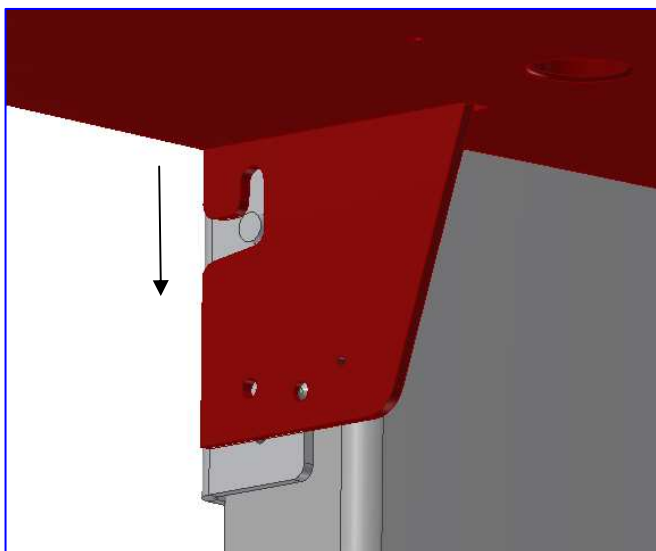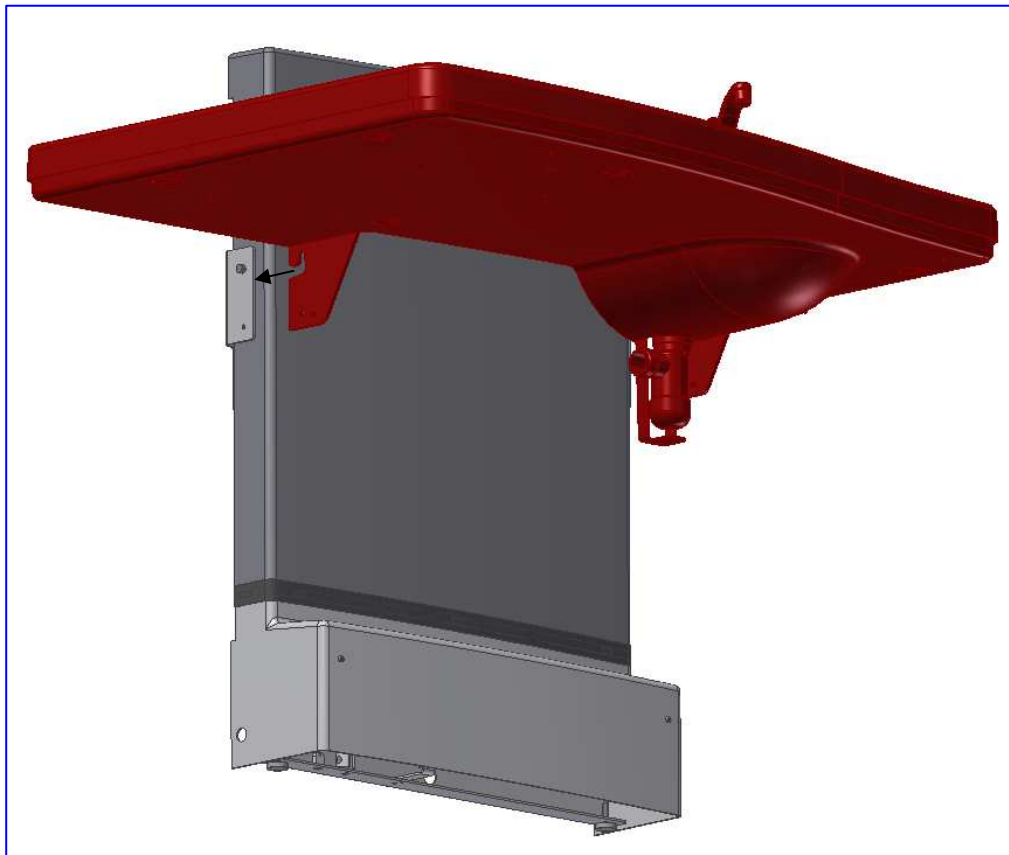





Mounting Instruction

5. Connection of the safety stop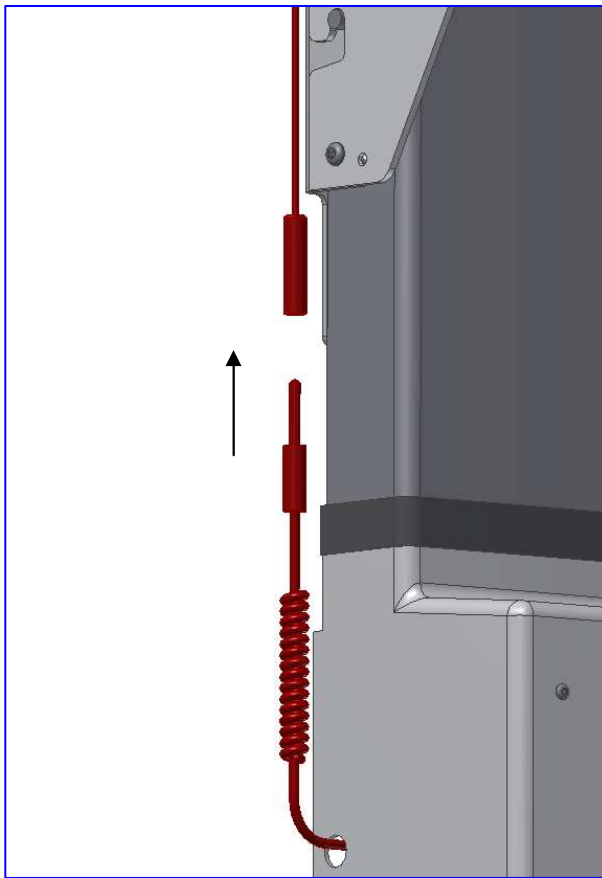


Mounting Instruction

6. Connection of water and drain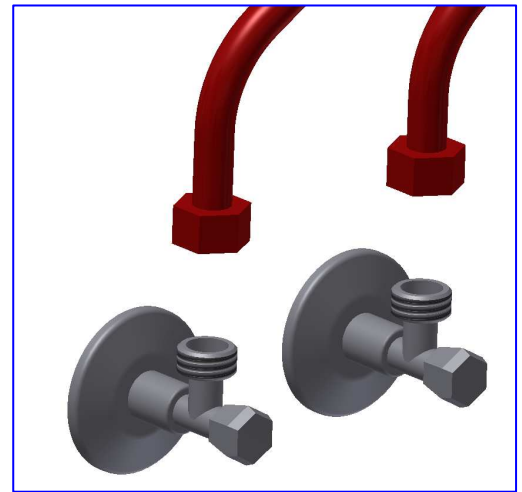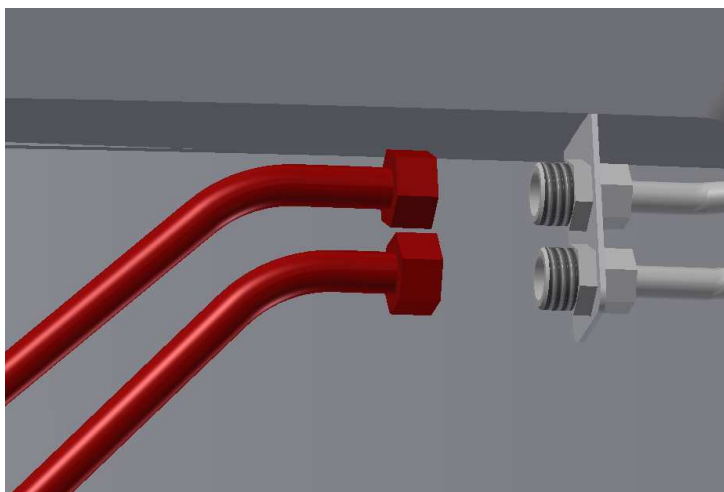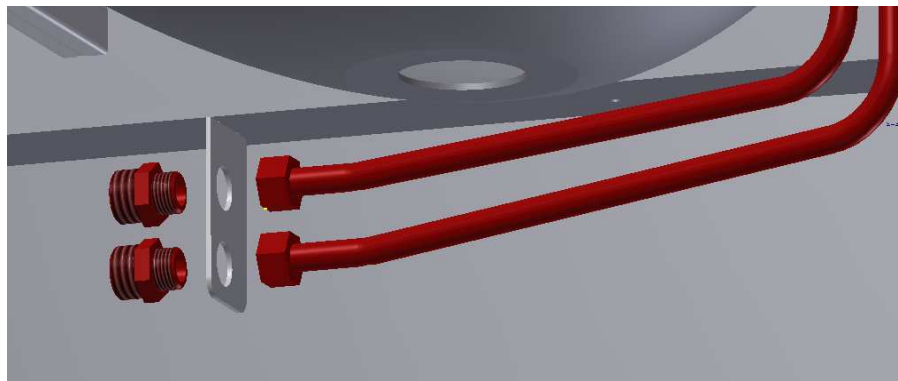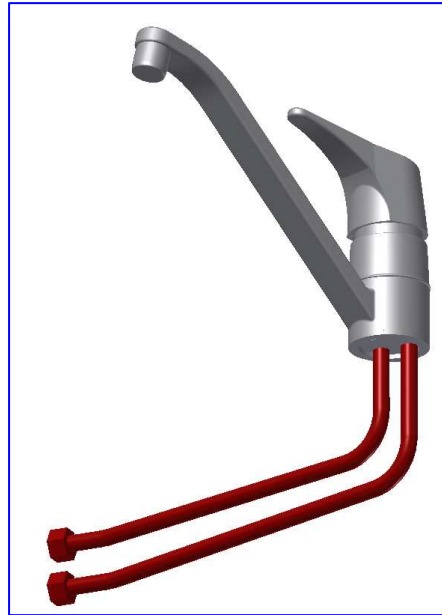

Mounting Instruction

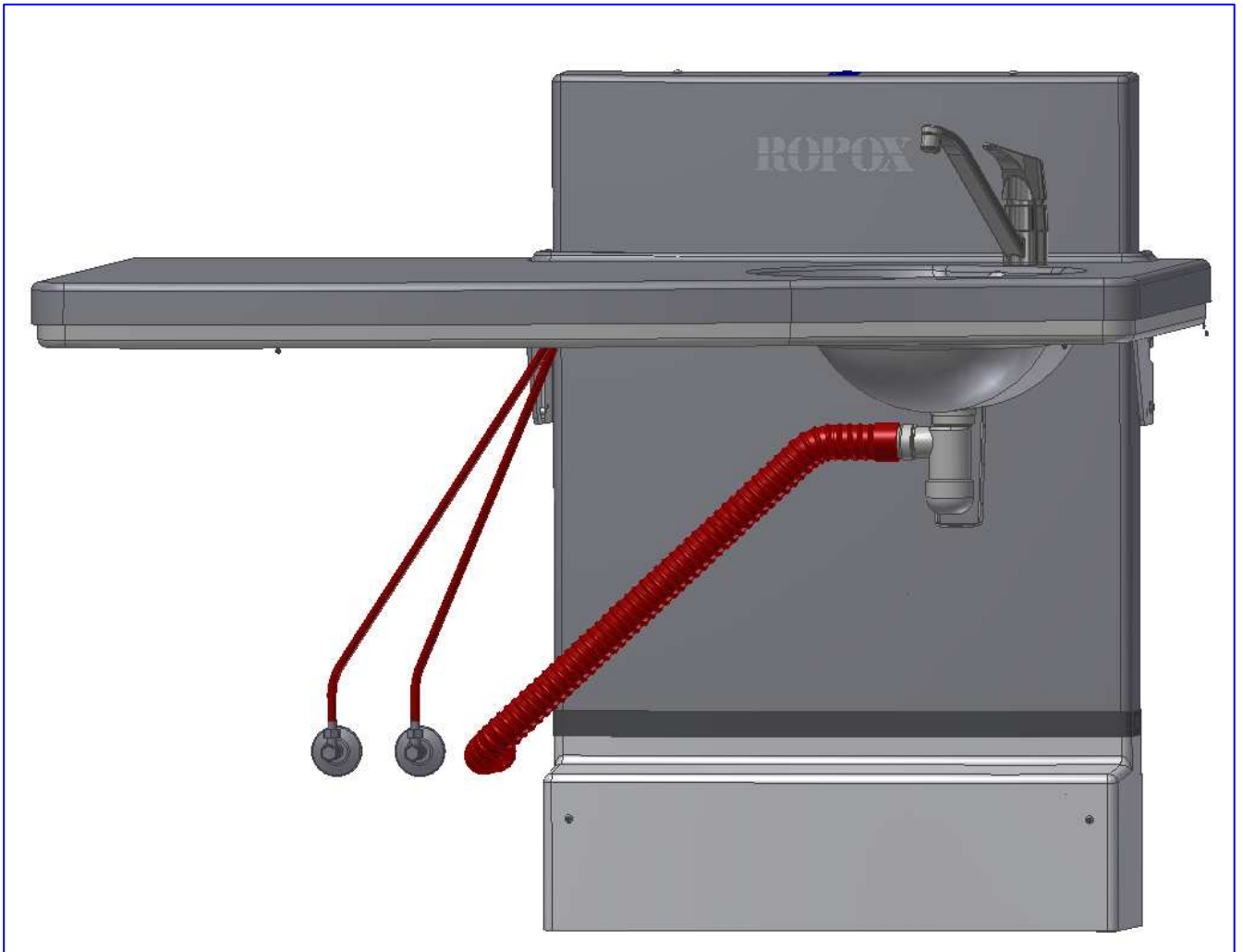Electrical components


Wiring circuit

The nursing table comes with control box mounted on the frame. The diagram shows how to connect the cables with the control box.

All plug-in connections are waterproof, and mounting/dismounting may therefore be a little difficult. Make sure that all plugs have the correct position before inserting them into the control unit. The plugs must be pressed home until the O-ring is no longer visible.

Motor and control box has an IPX6 rating. This means that they are dust-and waterproof.

The electrical system is designed for intermittent use, which means a maximum duty cycle of 10%; 2min on / 18min from.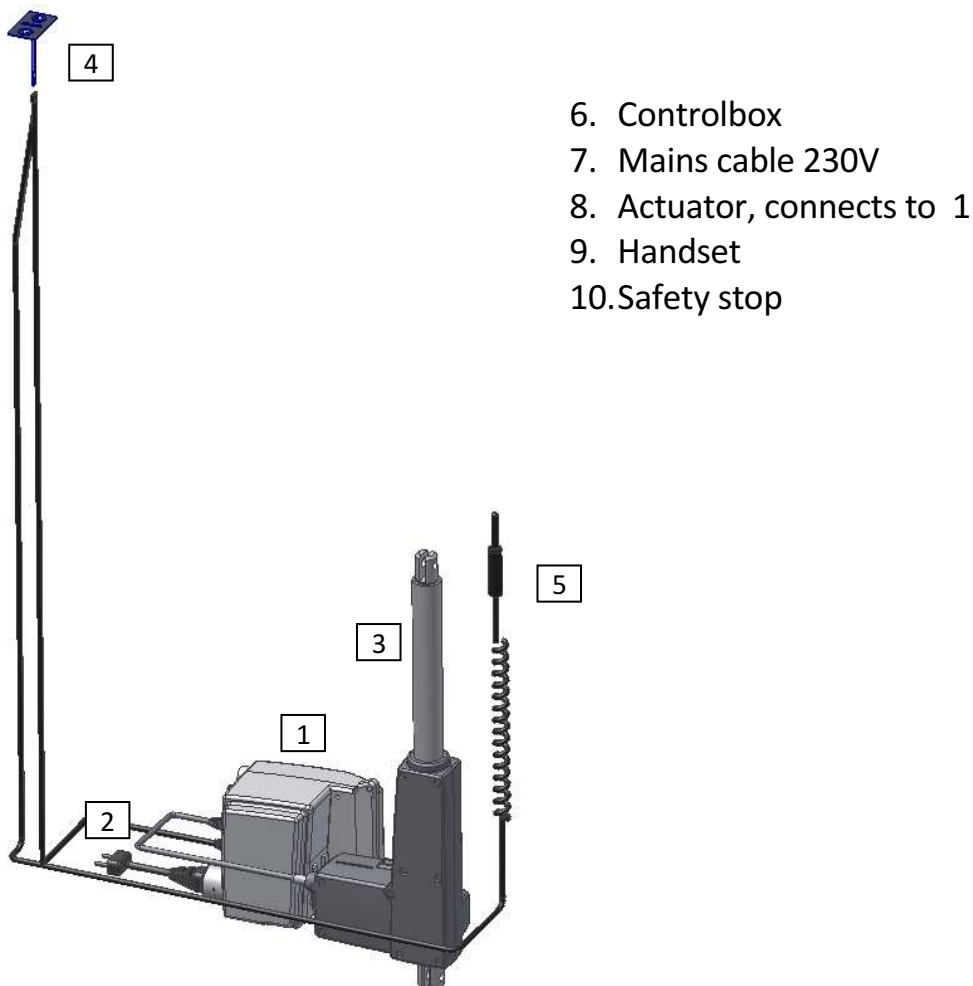


Electrical components

Connections

The nursing table comes with control box mounted on the frame.

Motor cable connected to port 1. Ports 2, 3 & 4 are not used and are therefore provided by blind plug.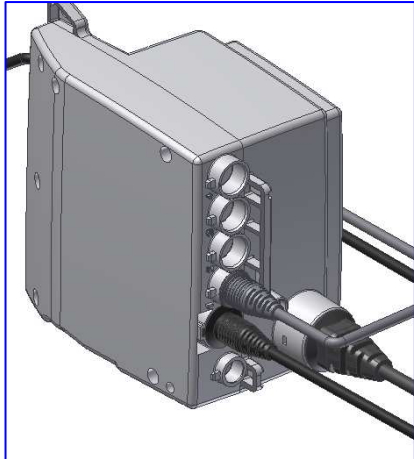

Handset must be connected to "HB".

The mains cable must be connected to the port on the side of the control box.

Dismounting


Just pull out the mains cable.

In order to maintain the full IPX6 protection class, it is very important that all cables have been mounted correctly.

Labelling

Label

Each nursing table has a label of its own as illustrated. This label contains important information to be stated in connection with inquiries to the supplier.

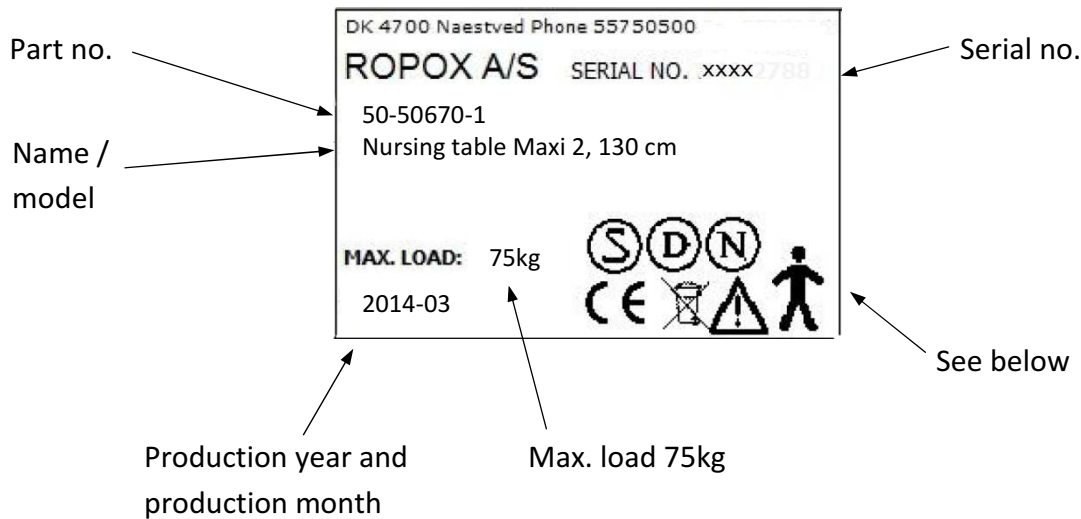Symbols:

User manual must be read before use.

Waste must be sorted - if possible for recycling.

Type B equipment.

Class 2 equipment. No connection to earth.

CE-marking. This product meets the requirements for General Safety and Functionality of the current EU Directives and Standards.

EMC information

The nursing table meets the requirements for Electromagnetic Compatibility (EMC), if installation and daily use are in accordance with the instructions in this manual. On rare occasions, the nursing table can be affected by electronic signals from e.g. laptops, mobile phones or the like.

Support

Troubleshooting

The nursing table does not move up and down when the hand-control switch is pressed	Check: <ol style="list-style-type: none">6. If 230V has been connected and current is available.7. If all plugs have been correctly inserted.8. If the table only moves upward, check if the safety stop has been blocked/activated.9. Dismount all cables and mount them again.10. Contact Ropox for support.
---	---

Waste

The used equipment must be disposed of in accordance with national law. We recommend disassembling of the product to the greatest possible extent, so that most parts may be recycled. Please note that this product contains electrical components. Examples of sorting groups: Metal, plastic, cables, electronics and parts for destruction.

The nursing table may be used at temperatures from 5°C to 45°C

The nursing table may be used at a relative humidity between 5% and 85% non-condensing

Factors affecting the product during storage and transport

Performance test

-  Ropox recommends testing the functions of the nursing table as well as retightening of all bolts and nuts once a year.

In case of defects, please contact Ropox as soon as possible and do not use the nursing table until the problem has been solved/remedied.

If you do not want to carry out this performance test yourself, contact Ropox to arrange a visit by a service technician. See contact information at the end of this manual.

Spare parts

97000420	Actuator Type : LA31.3 Stroke 300 mm	
97000434	Controlbox Type : CB9096-2	
97000244	Control switch	

Maintenance

The nursing table is maintenance-free. For safety reasons we recommend regular inspection of all bolts and moving parts to ensure correct fastening to the wall.

Cleaning

Do not use a high-pressure washer to clean this product.

Simply use a soft cloth or sponge with soapy water of max. 60 °C (pH 7) or cleaning alcohol.

Always clean the nursing table after use to prevent dirt from sticking to the surfaces.

**** IMPORTANT ****

Do not use scouring powders or other abrasives, as they will scratch the surface.

Also, avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkaline products.

Declaration of conformity

Ropox hereby declares under sole responsibility that the following products

50-50670 Ropox Nursing table Maxi 2, 130 cm

50-50672 Ropox Nursing table Maxi 2, 90 cm

are classified as Class I and comply with the following directives and standards:

DIRECTIVES


- Council Directive No. 89/392/EEC concerning Machinery, amended by Directive No. 98/37/EC
- 2006/95/EU Low Voltage directive, amended by directive No. 93/68/EEC
- 2004/108/EU, EMC-directive, amended by directive No. 92/31/EEC and 93/68/EEC
- DS/EN 12221-1:2008, Products for child care – nursing tables for private use – Part 1: Safety requirements

STANDARDS

DS/EN 1041: 2009
DS/EN 14971: 2008
DS/EN 12182: 2002

DS/EN ISO 60335-1
DS/EN 12221-1:2008

Date: 01-04-2014

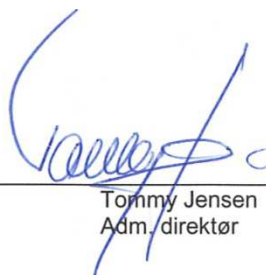Tommy Jensen
Adm. direktør

Claims

See General Terms and Conditions of www.ropox.com

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Ersatzteile

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97000420	Aktuator Typ : LA31.3 Hublänge 300 mm	 A black cylindrical actuator with a long silver shaft and a black mounting bracket at the end.
97000434	Steuereinheit Typ : CB9096-2	 A grey rectangular control unit with several ports on the front and a small circular feature on the side.
97000244	Ropox Handbedieneinheit	 A black rectangular hand-operated unit with two triangular buttons and the word 'ROPFOX' printed on it.

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Ropox A/S