

SOL-M™ Blunt Fill Needle

with and without filter



To help reduce the risk of needlestick injuries to healthcare workers when drawing up and preparing medication.

By eliminating the unnecessary use of sharps Blunt Fill Needle helps comply with Directive 2010/32/EU - Prevention from sharp injuries in the hospital and healthcare sector.

Easy Identification - Colour coded hub.

- Blunt Fill Needle (BFN) for use with vials.
- Blunt Fill Needle with 5 micron filter (BFNF) for use with glass ampoules.

Ease of Use

- Both needles have a conventional needle hub compatible with all standard syringes.

Reduction in Medication Contamination

- The 5 micron filter in the hub of the BFNF means any glass particles larger than 5 microns are filtered out preventing them from being drawn into the syringe prior to injection.





SOL-M™ Blunt Fill Needle

with and without filter

Medication Preparation

Usage Guidelines

Remove the Blunt Fill Needle from its package and connect to the syringe.

Remove the needle cap.

BFN - Insert the needle into the centre of the vial stopper at a 90° angle. This will help reduce the risk of rubber fragments from the side of the vial stopper contaminating the medication.

BFNF - Draw up medication from glass ampoule.

Once medication has been prepared and is in the syringe ready for administration, remove the BFN/BFNF and dispose of in a sharps container.

The Blunt Fill Needle is for **single use only**. Reuse will dull the needle, remove the silicone coating and increase the chance of coring and rubber fragmentation.



18 Gauge
45° Bevel



- Red – Blunt Fill Needle
- Purple – Blunt Fill Needle with Filter

Reference number	Product description	Gauge size	Outer diameter (mm)	Length	
				(mm)	(inch)
110023	SOL-M™ Blunt Fill Needle 18G*2"	18G	1.2 mm	50 mm	2"
M110023F	SOL-M™ Blunt Fill Needle w/ Filter 18G*2"	18G	1.2 mm	50 mm	2"
110022	SOL-M™ Blunt Fill Needle 18G*1 1/2"	18G	1.2 mm	40 mm	1 1/2"
M110022F	SOL-M™ Blunt Fill Needle w/ Filter 18G*1 1/2"	18G	1.2 mm	40 mm	1 1/2"
110021	SOL-M™ Blunt Fill Needle 18G*1"	18G	1.2 mm	25mm	1"
M110021F	SOL-M™ Blunt Fill Needle w/ Filter 18G*1"	18G	1.2 mm	25 mm	1"

Sales Unit: 100 pcs/Box; 1000 pcs/Case

For more information, please contact:

Sol-Millennium Europe Sp. z o.o.
Twarda 18, PL-00105 Warsaw, Poland
Tel: +48 22 202 67 23, e-mail: info.EU@sol-m.com

sol-m.com

2018-03-BFN-0225