RK1025
RK2924-30

Indivo kitchen – Operation and maintenance manual	en
Indivo Hyldeløfter – Brugs- og vedligeholdelsesvejledning	dk
Indivo kök – Bruks- och underhållsanvisning	sv
Indivo kjøkken – Bruks- og vedlikeholdsveiledning	no

English	3
Dansk	13
Svenska	23
Norsk	33

Pressalit's Environmental and Quality Management Systems are certified by LRQA

- en** The warranty covers faults or defects in the material of our products within a period of 3 years from date of invoice. Products subject to minor technical modifications and design deviations E. & O.E.
-
- dk** Garantien dækker i 3 år fra fakturadato for materiale- og fabrikationsfejl. Der tages forbehold for trykfejl, tekniske ændringer og modelafvigelser.
-
- sv** Garantin gäller i 3 år från fakturadatum för material- och fabrikationsfel. Med reservation för tryckfel, tekniska ändringar och modellavvikelser.
-
- no** Garantien gjelder i 3 år fra fakturadato for material- og fabriksjonsfeil. Med forbehold for

Content

Mounting instruction	3
Safety	4
Usage	4
Safety labels.....	5
Operating instructions	6
Safety system.....	7
The cupboard safety plates	7
Maintenance	8
Resetting.....	8
Cleaning.....	8
To remove limescale.....	8
Dimensions (mm)	9
Technical data.....	10
Disposal and recycling	10
Trouble-shooting.....	11

Mounting instruction

en

A mounting instruction is included with the product. It can also be downloaded from www.pressaliticare.com.

Pressalit cannot be held liable if the product is used in any way which differs from that described in this guide.

We reserve the right to amend this manual without prior notice.

Original instructions

Safety

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Never use or handle this product in other ways than are specified in this manual.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.
- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.



Usage

The height adjustable lifting unit allow both seated and standing users to work comfortably in the same kitchen. The unit is for indoor use only.


en **Max load is 40 kg inclusive shelves.**

Safety labels


Before initial operation, please affix the safety labels supplied along with this booklet.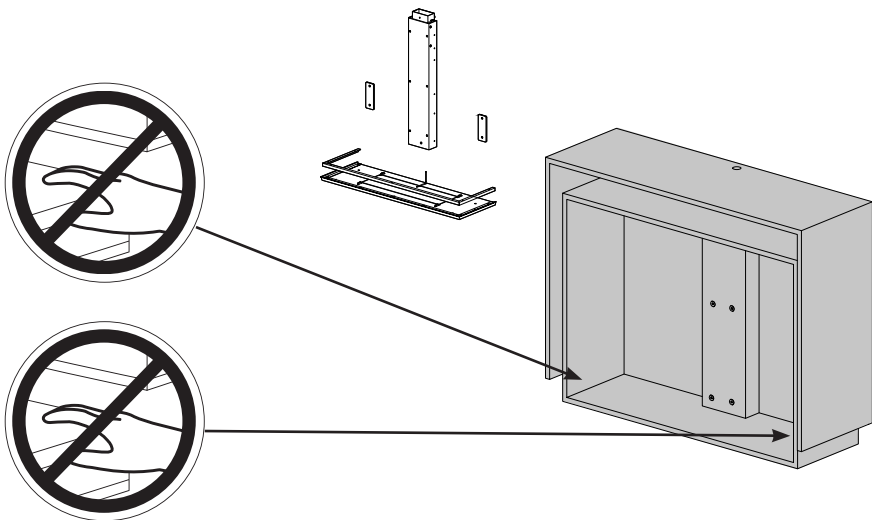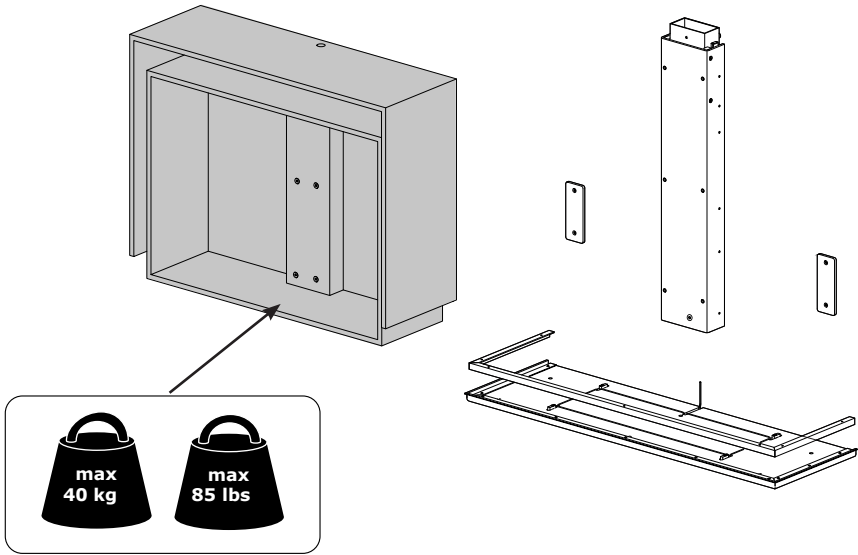


Operating instructions

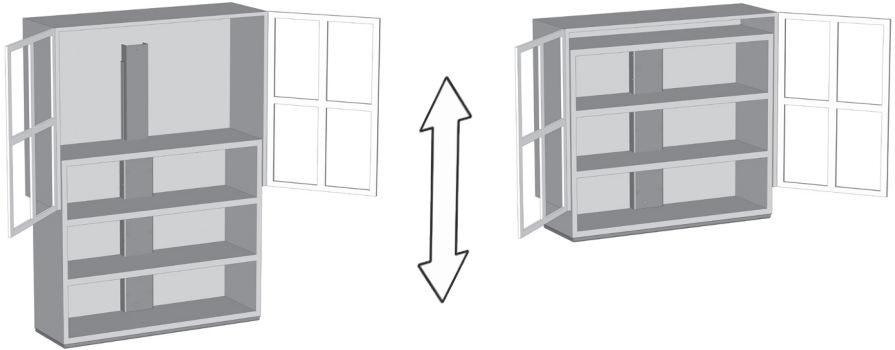
The height adjustment is controlled by operating the buttons on the fascia of the worktop. The cupboards can be adjusted 430 mm up and down.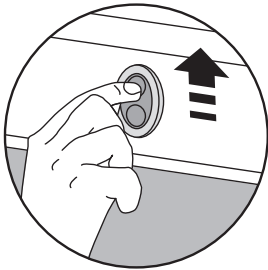

Press and hold the up button to raise the cupboards, and press and hold the down button to lower them. When the button is released, the motion stops.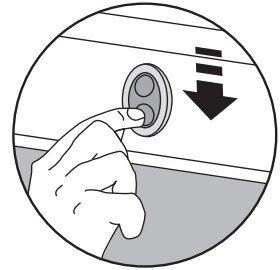

en

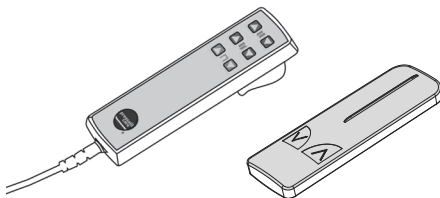An optional hand-held controller may be purchased as an alternative or supplement to the operating buttons. The controller is available in a wired and a wireless version.


Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

Safety system

The lifting unit is equipped with a safety system to prevent crushing injuries.

The cupboard safety plates


If any of the safety plates under the lifting unit come in contact with an object when the shelves are being lowered, the motion stops immediately and the shelves move up 10 mm.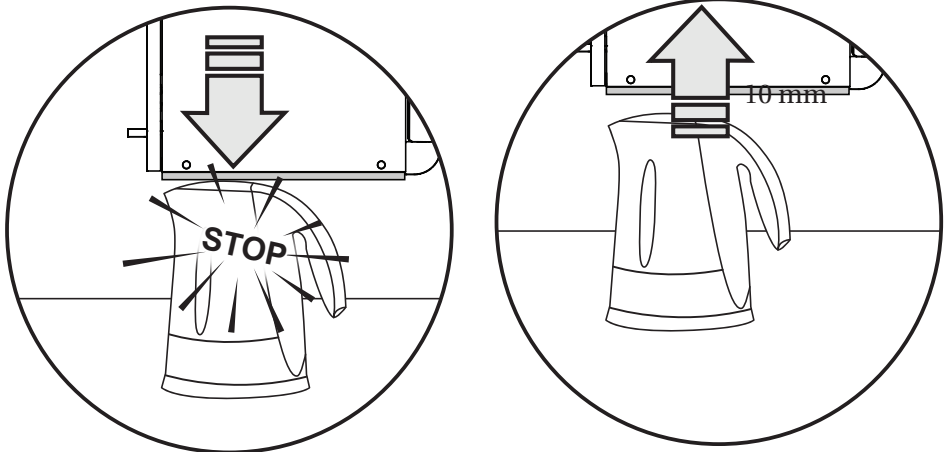

en

Maintenance

It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the product to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.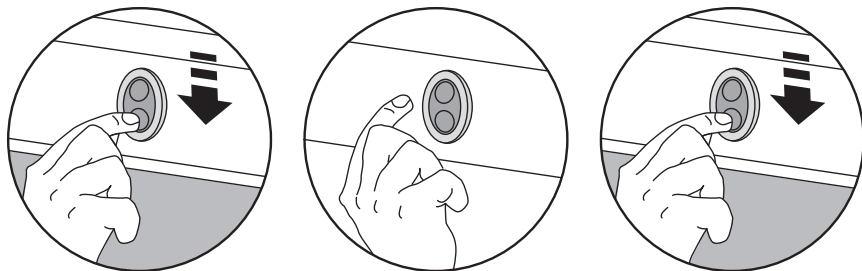

Resetting

It is also recommended on a weekly basis to reset the lifting mechanism.

- Hold in the switch until the shelves stops in the lowest position.
- Release the switch.
- Press and hold in the switch once more for 5 seconds.

en

Cleaning


Use ordinary soapy water and dry with a soft cloth. Avoid the use of abrasive, etching or chloride cleaning agents.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts of 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible.

Dimensions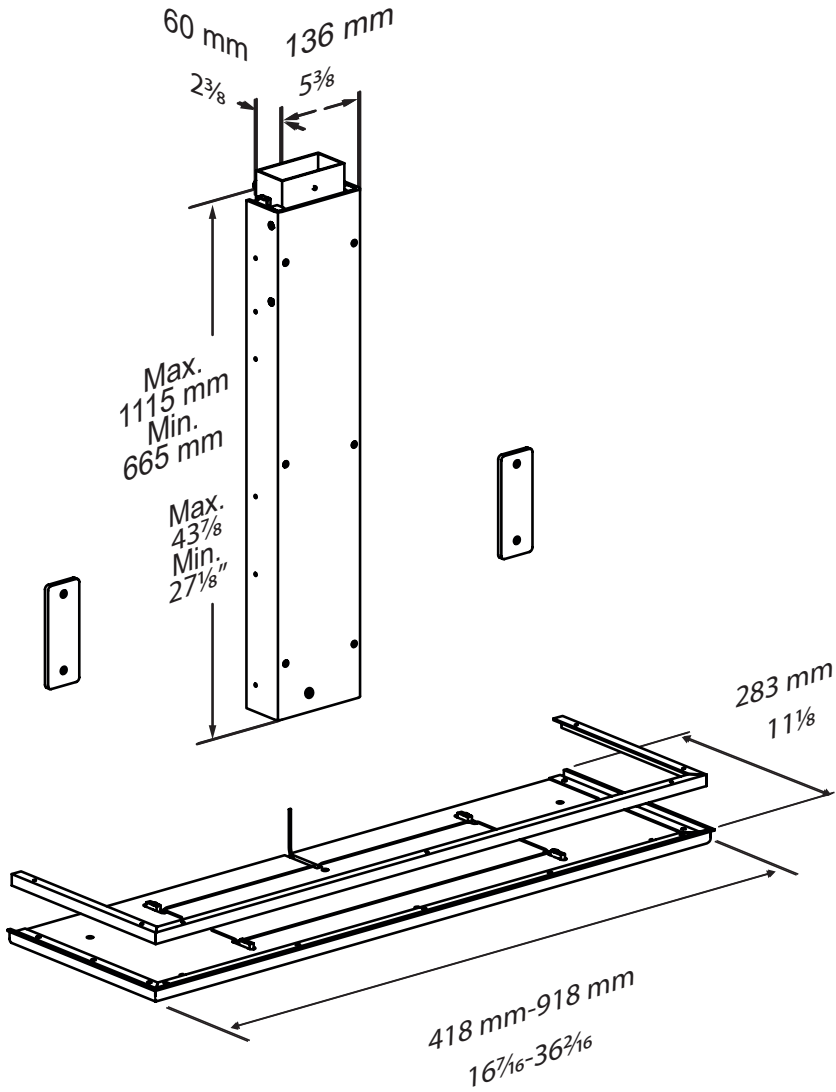

en

Technical data

Name	
Max load (incl. shelves)	40 kg / 88 lbs
Materials	Steel
Weight	38 kg / 83.77 lbs

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

en


Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

Trouble-shooting

Symptom	Cause	Action
Operating buttons don't work.	Fault in the power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit A/S or your local dealer.
	The unit has not been reset. (For instance after a power outage).	Reset the unit as described under "Resetting".
	Faults on the connectors	Check the plugs from the motor to the extension cord and the connectors in the control box inputs. The male connectors have some very small "steel legs" which can easily be bent when inserted into the control box. If a "steel leg" is bent, correct it before re-inserting it in the control box. Remember to reset the lifter.
	The fascia switch cannot be operated.	Check if the fascia contact can be operated unobstructed and if it returns to neutral position. Movement may be hampered by dirt or grease. Check the socket connector: Check that all wires are properly installed according to the installation instructions and that none of them are broken. If a cord is not properly mounted, the lifting device can only run down. NOTE: If the lifting unit is lowered, the contact is out of order. If all wires from the connector are properly mounted and none of them are broken, the lifting device must be able to move up / down. Remember to reset the lifter.

en

Symptom	Cause	Action
Safety plate doesn't work.	Fault in the safety system.	Contact supplier.
Product runs up and down unevenly	Lubrication needed	Take the lifting unit to its lowest position. Lubricate the slide rails ① behind the lower cupboard with WD-40 or equivalent.

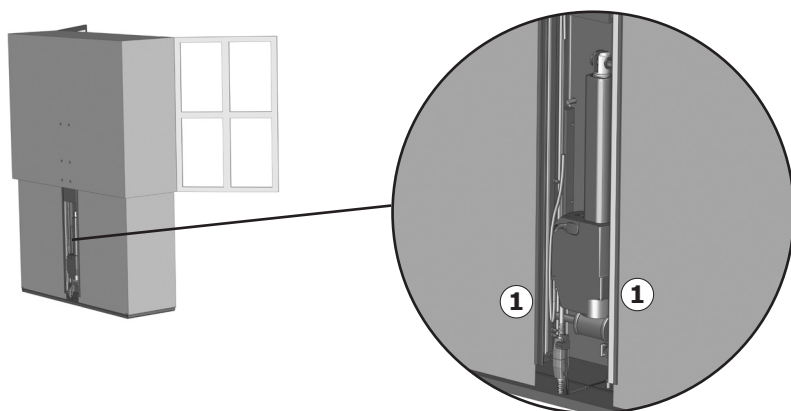en

Contact Pressalit Care or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

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